

Sontec Instruments®

VASECTOMY SET

FINEST QUALITY IN THE INDUSTRY / AGGRESSIVE PRICING



NO SCALPEL / puncture method / less discomfort / virtually bloodless / quicker healing / fewer side effects / low to no risk of hematoma / promoted by the World Health Organization

Vasectomy Fixation Clamp

516-700 3 mm ring 5 1/2" (14 cm)
516-702 5 mm ring 5 1/2" (14 cm)

* To isolate vasectomy in the scrotum, withdraw it through the incision and secure it prior to clipping

Vasectomy Dissection Forceps

516-720 curved 5" (12.5 cm)

* With sharp tips to puncture the scrotum and dissect the vasectomy

Goldman-Fox Scissors, PC gold

1200-405 curved 5" (12.5 cm)

Vasectomy Hook

516-715 sharp 5 1/4" (13 cm)

Sets -----

Vasectomy Set #1

516-710 includes: hook, clamp, forceps

Vasectomy Set #2


516-725 includes: hook, clamp, forceps, scissors, 7 1/2" x 4" x 3/4" instrument tray

Vasectomy Set #3

516-730 includes: hook, clamp, forceps, 7 1/2" x 4" x 3/4" instrument tray

Vasectomy Set #4

516-731 includes: hook, clamp, forceps, scissors
* Also available separately, Instrument Tray, with silicone mat 1103-723 7 1/2" x 4" x 3/4"

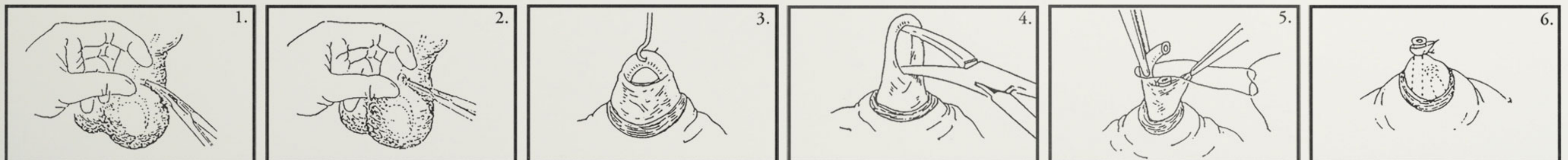

1. Using the Vasectomy Dissection Forceps (516-720), forcibly puncture the scrotum and spread, leaving a small opening.

2. Insert the Vasectomy Fixation Clamp (516-700), grasp the vas and withdraw through the incision.

3. Hook the vas with the Vasectomy Hook (516-715)

4. Dissect the vas with the Vasectomy Dissection Forceps (516-720)

5. Tie the ends with clips or cautery and return the vas to the scrotum



AUTHORIZED DEALER

A TRADITION OF FINE SURGICAL INSTRUMENTS

Colorado / USA / 800.821.7496

www.SontecInstruments.com