

INSTRUCTIONS FOR USE - Convertible Kerrison Rongeur



- ALWAYS USE TWO HANDS -

Closed Locked Position: Release to Clean

1. Squeeze handles together.
2. Push trigger down into open position.
3. Lift up on shaft to clean inside.
4. Remove any blood or tissue with soft brush and rinse.



Closing Procedure:

1. Align upper shaft flange into lower shaft groove while in open position.
2. Once aligned, squeeze handles together and push trigger up into locked position.
3. Test instrument for proper function after locking together.



Sontec's instruments are guaranteed to be free of defects in materials and workmanship. Any Sontec instrument that is determined to be defective will be replaced or repaired at our discretion. **Instruments not properly cared for, used for purposes not intended or used beyond the design capacity will NOT be covered.** Our liability under this guarantee is limited to the repair or replacement of the defective merchandise.