

# Sontec Instruments®

## VASECTOMY SET

FINEST QUALITY IN THE INDUSTRY / AGGRESSIVE PRICING



NO SCALPEL / puncture method / less discomfort / virtually bloodless / quicker healing / fewer side effects / low to no risk of hematoma / promoted by the World Health Organization

### Vasectomy Fixation Clamp

516-700 3 mm ring 5 1/2" (14 cm)  
516-702 5 mm ring 5 1/2" (14 cm)

\* To isolate vasectomy in the scrotum, withdraw it through the incision and secure it prior to clipping

### Vasectomy Dissection Forceps

516-720 curved 5" (12.5 cm)

\* With sharp tips to puncture the scrotum and dissect the vasectomy

### Goldman-Fox Scissors, PC gold

1200-405 curved 5" (12.5 cm)

### Vasectomy Hook

516-715 sharp 5 1/4" (13 cm)

### Sets -----

#### Vasectomy Set #1

516-710 includes: hook, clamp, forceps

#### Vasectomy Set #2


516-725 includes: hook, clamp, forceps, scissors, 7 1/2" x 4" x 3/4" instrument tray

#### Vasectomy Set #3

516-730 includes: hook, clamp, forceps, 7 1/2" x 4" x 3/4" instrument tray

#### Vasectomy Set #4

516-731 includes: hook, clamp, forceps, scissors  
\* Also available separately, Instrument Tray, with silicone mat 1103-723 7 1/2" x 4" x 3/4"

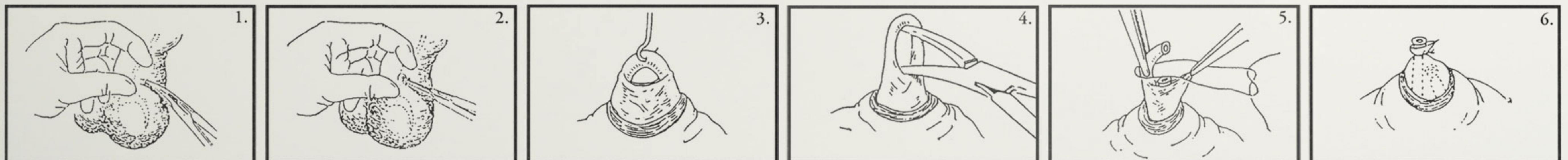

Using the Vasectomy Dissection Forceps (516-720), forcibly puncture the scrotum and spread, leaving a small opening.

Insert the Vasectomy Fixation Clamp (516-700), grasp the vas and withdraw through the incision.

Hook the vas with the Vasectomy Hook (516-715)

Dissect the vas with the Vasectomy Dissection Forceps (516-720)

Tie the ends with clips or cautery and return the vas to the scrotum



AUTHORIZED DEALER

A TRADITION OF FINE SURGICAL INSTRUMENTS

Colorado / USA / 800.821.7496

www.SontecInstruments.com